
Neogen Corporation
944 Nandino Blvd • Lexington, KY 40511
800-621-8829 • 859-254-1221


animalsafety.neogen.com

INSTRUCTION MANUAL

PRIMA-SHOT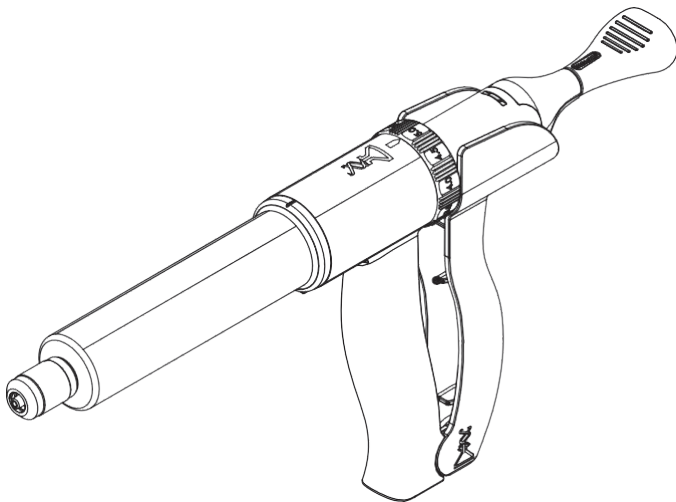

PRIMA-SHOT

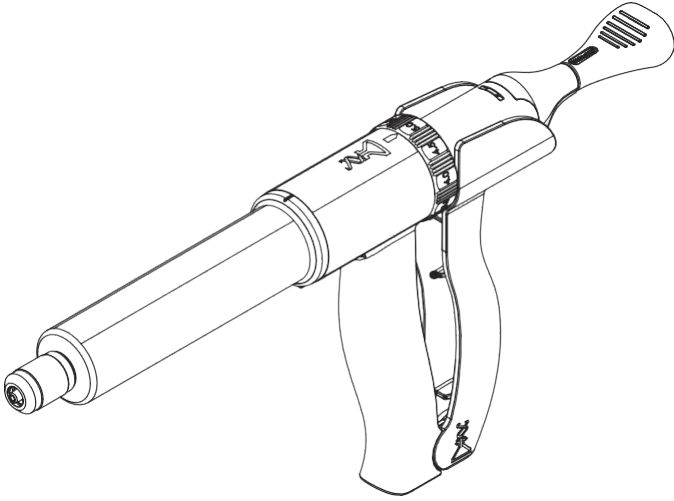
CONTENTS

English	Page 1
Spanish	Page 4
French	Page 7


INSTRUCTIONS FOR USE

TO FIT NEEDLE WITH METAL LEUR NEEDLE NUT: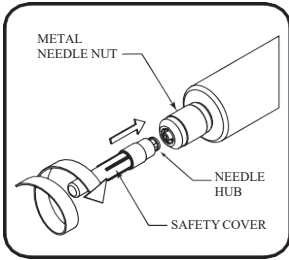

- 1.) For safety, leave protective cover on needle until ready to prime the instrument.
- 2.) Do not remove the needle nut to mount or dismount the needle.
- 3.) Make sure metal needle nut is finger tight; do not tighten with tools, as this can cause damage to the threaded tip of the barrel
- 4.) Push the hub of the needle into the needle nut and turn clockwise until the needle is tightly secure.
- 5.) When you are ready to prime the instrument, remove the needle cover by turning clockwise and pulling the cover at the same time. The cover will twist off, exposing the needle. Keep the needle cover for a safe needle removal after use.

To remove the needle, replace the safety cover, and turn the needle counter-clockwise. The needle should twist out of the needle nut. Discard the needle in bio-safe container.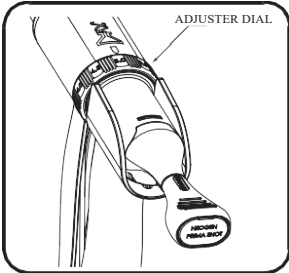


TO SET DOSAGE:


- 1.) For safety, leave protective cover on the needle until ready to prime the instrument.
- 2.) Rotate the Adjuster Dial so that the desired dosage size in ml's is aligned with arrow.

Setting the dosage amount can be done at any time except while administering an injection.

TO FILL BARREL: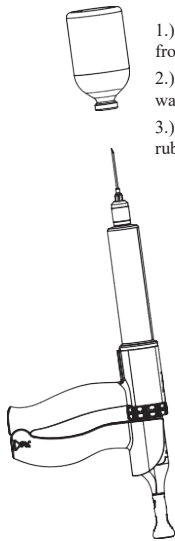

- 1.) Remove safety cover from needle.
- 2.) Ensure piston is all the way forward.
- 3.) Insert needle through rubber stopper into bottle.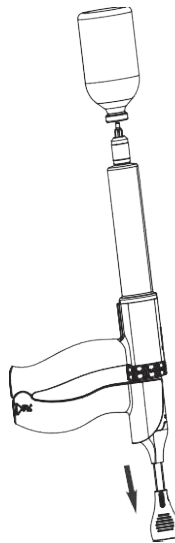

- 4.) Rotate knob counter-clockwise 90 degrees.
- 5.) Grasp the plunger pull knob and pull away from the barrel gradually, until you reach the desired volume or the barrel is full.
- 6.) Rotate knob clockwise until it stops. The rear handle is now engaged with the plunger.

PULL KNOB
UNLOCKED


7.) Remove bottle from needle.

8.) If an air bubble remains in the barrel, point the barrel upward allowing the bubble to float to the needle end.

9.) Set the dose to 0.5ml, then carefully squeeze the handles to expel the air without pushing the fluid out.


10.) Once the air is expelled out of the barrel, ensure desired dosage is selected and then the syringe is ready to use.

COMPONENTS OF THE PRIMA SHOT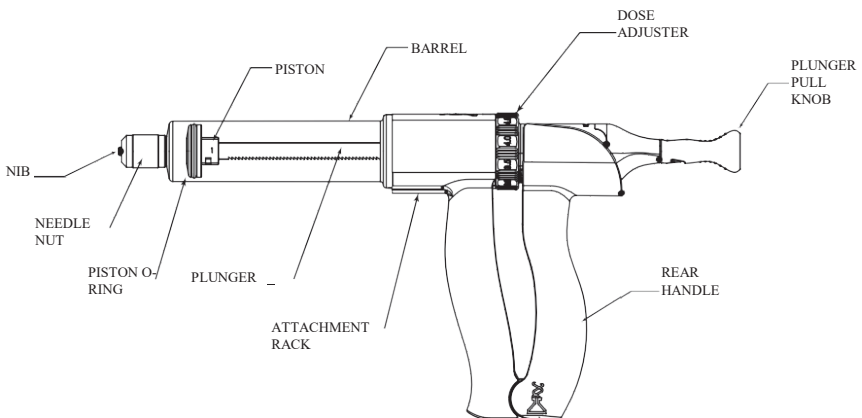


CLEANING INSTRUCTIONS

- 1.) If a needle is installed, remove it.
- 2.) Grasp needle nut firmly and rotate it counter-clockwise until it detaches.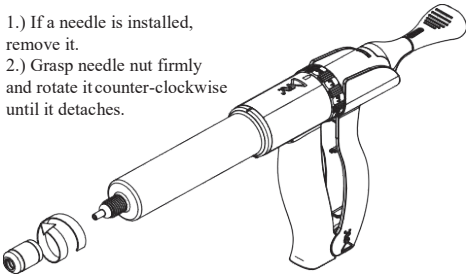

- 3.) Grasp barrel firmly and rotate it counter-clockwise until it detaches.

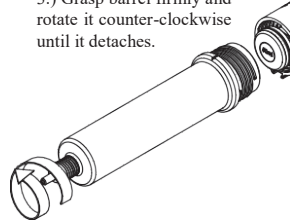4.) Place all the disassembled components into a container of warm, soapy water and let them soak until any grime and debris loosens.

5.) Remove components one at a time and lightly scrub with a soft bristle brush to remove any remaining debris still clinging to the component. If stains or debris remain on the component, place it back into the soapy solution and let it soak further until all contaminants can be safely removed without excessive force when scrubbing.

6.) Under warm running tap water, rinse off the soapy residue left on each component and set aside on a towel.

7.) Soapy liquid will remain inside the handle chassis; take care to hold the chassis so that the running water can run inside it and flush out all the soapy residue.

8.) Gently shake out as much rinse water as you can from the chassis then set aside on the towel in a vertical position so that any remaining liquid can drain. Take each component and hand wipe to complete drying those parts. Use a rolled up towel inserted in the barrel and rotate it to remove clinging moisture from inside the bore.

9.) When the handle chassis appears to have most or all of the moisture drained, towel dry the chassis by hand.

10.) Reassemble the syringe by reversing the above disassembly instructions.

LUBRICATION

If vaccinator becomes sluggish, clean thoroughly using the cleaning instructions. You will need to remove the barrel assembly, clean and apply Prima Tech lubricant to the piston o-ring.

Many times, product failure is due to insufficient cleaning (dried vaccine in valves, bung material caught in draw spike, etc.). For complete cleaning instructions, please see above cleaning instructions.

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